

# Field testing guide

To do

**⚠ WARNING : This bandage was designed based on existing israeli bandages, however it wasn't certified as a medical device and should thus be used based on your own jugment. ⚠**

Table 1: Suggested application procedure from our limited testing :



**Step 1.a :** Clip the lever on the compressive bandage, while holding pressure on the victim's wound (if they are conscious they can apply pressure themselves).


**Step 1.b (alternative) :** Apply the bandage to the wound and do a single turn around the victim's limb.

**(using velcro) :** if you are using Velcro, you can attach the lever directly over the wound now.




**Step 2.a :** Do a single turn around the victim's limb and slide the lever over the wound.

**Step 2.b (alternative) :** Put the lever directly over the wound before finishing the first turn and pass the bandage through the inner passage.

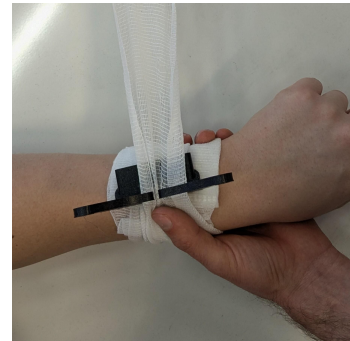

**Step 3 :** Do another turn around the limb and pass through the normal lever passage.

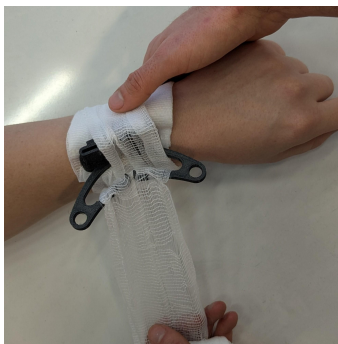

**Step 4 :** You can do an additional turn, and then you have to start winding the other way around the limb to flip the lever back to generate pressure.



**Step 5 :** Keep winding around the limb to firmly hold the lever into place.



**Step 6 :** Once you have finished you can simply fix the end of the bandage into a tight part of the bandage to hold it in place.